


When Managing Bioburden is a Priority

- A unique antibacterial wound dressing that can be used under Ostomy barriers
- Manages bioburden, which helps the bodies own immune system restore bacterial balance
- Broad spectrum activity against a wide variety of bacteria and yeast commonly found in wounds
- No hydration required

Hydrofera **BLUE**®
■■■■

Site Preparation

- Prior to the initial preparation of the Hydrofera Blue Ostomy dressing, the wound should be debrided of any necrotic tissue
- The wound should be cleansed with normal saline or an appropriate cleansing solution

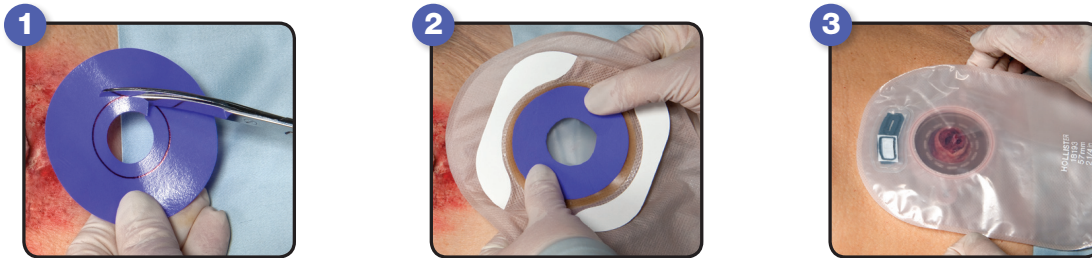

Application Instructions

- 1 Trim the inner diameter of the Hydrofera Blue Ostomy dressing to fit around the stoma. If needed, trim the outer edge of the dressing as well, so there is at least 1/2" of the skin barrier extending beyond the outer edge of the dressing.
- 2 Apply the film side of the dressing to the Ostomy skin barrier.
- 3 Apply the skin barrier to the skin.

A new Hydrofera Blue Ostomy dressing should be applied when the skin barrier is changed. Hydrofera Blue dressings should be changed every 1-3 days depending on the level of exudate and the condition of the wound.

For extra security holding the skin barrier in place:

- Consider using an Ostomy belt or skin barrier with tape border
- Consider using a larger skin barrier to ensure coverage over the entire wound area, and so there is adequate skin barrier around the outer circumference of the Hydrofera Blue Ostomy dressing

For product questions, sampling needs, or detailed clinical questions concerning our products call **860-337-7730**.

Hydrofera Blue Ostomy Dressing

Product No	Product Size	Quantity/Box	HCPCS CODES*
HBRF2650	2.5" (6.4 cm) Diameter Ostomy Dressing	10	A6209

Hydrofera Blue Dressings also available:

Product No	Product Size	Quantity/Box	HCPCS CODES*
Standard Dressings			
HB2214	2" x 2" (5 cm x 5 cm)	10	A6209
HB4414	4" x 4" (10.2 cm x 10.2 cm)	10	A6209
HB4514	4" x 5" (10.2 cm x 12.7 cm)	10	A6210
HB6614	6" x 6" (15.2 cm x 15.2 cm)	10	A6210
HBT0906	9-mm Tunneling Dressing (1.2 g)	10 pair	A6215
Heavy Drainage Dressings			
HBHD4450	4" x 4" x 0.5" thick (10.2 cm x 10.2 cm x 1.2 cm)	5	A6209
HBHD6675	6" x 6" x 0.75" thick (15.2 cm x 15.2 cm x 1.9 cm)	5	A6210

Hydrofera®

www.hydrofera.com

Hydrofera Blue Dressings
Manufactured By:
Hydrofera, LLC
340 Progress Drive
Manchester, CT 06042 USA

*REIMBURSEMENT DISCLAIMER: The Centers for Medicare & Medicaid Services (CMS) uses Palmetto GBA as the Pricing, Data Analysis, and Coding (PDAC) Contractor. The PDAC Contractor maintains the Product Classification List, located at https://www4.palmettogba.com/pdac_dmecs/. The Product Classification List is a searchable database containing products that have received HCPCS codes verification from PDAC and their corresponding HCPCS codes. Hydrofera LLC, however, makes no representations as to the accuracy of the information contained within the PDAC database, nor any representations as to whether its products are reimbursable under any government sponsored healthcare program and/or private-insurance program. It is the provider's sole responsibility, in consultation with the insurer, if necessary, to determine medical necessity, ensure coverage criteria is met, and submit appropriate codes, modifiers, and charges for services rendered.

Hydrofera Blue and the Hydrofera Blue logo are trademarks of Hydrofera, LLC.
©2023 Hydrofera, LLC. Hydrofera LLC is an Essity company.
Made in USA.

14-001-005 Rev. 01 65145 R1 A24